Lunos^{eye}

DE Gebrauchsanweisung | EN Instructions for Use
FR Notice d'utilisation | IT Istruzioni per l'uso
ES Instrucciones de uso | SE Bruksanvisning
DK Bruksanvisning | NO Bruksanvisning
NL Gebruiksaanwijzing

Lunos Eye

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DE Deutsch

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Lunos Eye

Instructions for Use

EN English	
Product Eye cover for shielding light in newborns and infants aged 0 to 6 months .	Intended purpose Lunos Eye is an eye cover designed to shield light in the eye area of newborns and infants aged 0 to 6 months. The product is intended for use only as specified and does not constitute a medical treatment or medical device.
Before use Check the packaging and product for damage or loose parts before each use. Use only an undamaged product. Before application, ensure that the airways (nose and mouth) remain unobstructed at all times.	Directions for use  1. Check the product for integrity before use 2. Slightly lift the head and position over the eyes without applying too much pressure 3. Close the fastening. Ensure that the nose and mouth remain free and that no pressure is applied to the eyes
After use Remove the product carefully. If necessary, clean according to the care instructions. Dispose of in household waste at the end of its service life..	Warning notice Use only under constant adult supervision. Never leave newborns or infants unattended. Airways (nose and mouth) must remain unobstructed at all times. Do not apply too tightly. Do not apply pressure to the eyes. Check the product and packaging for damage or loose parts before each use. Do not use the product if it is damaged or if parts may become loose. Discontinue use immediately in case of restlessness, breathing difficulties, or skin reactions and remove the product. Do not use during unsupervised sleep. Keep out of reach of young children. Not a toy.
Material 92% polyamide; 8% spandex	Care instructions Check for cleanliness and integrity before each use. If soiled, the product may be carefully cleaned with a slightly damp cloth. Allow to dry completely before reuse. Do not use harsh cleaning agents. Do not continue use if the product is damaged or parts become loose.
Symbols	
 Product reference / Article number	 Use-by date
 Batch number	 Consult instructions for use
 Keep away from heat	 Keep dry
 Do not use it if packaging is damaged	 Importer
 Single patient – multiple use	 Temperature limits
 Manufacturer	Manufacturer Fenik GmbH Rossbergstraße 2 72379 Hechingen-Boll Germany hello@fenik.de

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